

Cleaning and Disinfection of the Dialysis Station

Checklist
#4

Notes: All items listed below must be disinfected using an EPA-registered tuberculocidal disinfectant prepared and used per manufacturer's instructions for use; Staff PPE must be gown, face shield or mask/eye protection, gloves

- Remove all bloodlines & disposable equipment & discard in biohazardous waste; dialyzer for reprocessing cap all ports; transport dialyzer in a manner to prevent contamination of other surfaces
- Remove gloves; hand hygiene; don clean gloves
- Obtain wetted disinfectant cloths/wipes
- Use disinfectant cloth/wipe to visibly wet all machine top, front and side surfaces, dialysate hoses, Hansen connectors, and outside surfaces of concentrate jugs
- Empty prime waste receptacle; disinfect all internal and external surfaces
- When chair is vacated, remove and discard disposable supplies or dedicate to that patient-not returned to common supplies
- Recline chair fully, use a fresh disinfectant cloth/wipe to visibly wet all external front-facing and side chair surfaces, including along sides of seat cushion and side tables
- Apply disinfectant to all non-disposable items: BP cuff and tubing, TV controls, call button, data entry station & counters around station
- Discard cloths/wipes
- Remove gloves; hand hygiene